SCHAPER

Manufacturer

Orthopädie Schaper GmbH
Dr.-Salzmann-Straße 6
4600 Wels
Austria
office@schaper-wels.at
www.bandagist-schaper.at

Distribution

Schaper Benelux
Rivierdijk 226
3372 BN Hardinxveld
Nederland
ronald@schaper-benelux.com

DEU

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4**UK**

Active bandage shaped anatomically by knitted fabric and with integrated padding to reduce strain on the nerves and blood vessels of the wrist. Very elastic and breathable materials ensure compression (enhances healing) and very comfortable to wear. Individually adjustable stabilisation due to its adjustable bar insert on the palmar side (also removable). Extra Velcro strap supports stabilising action.

Applying the Bandage *See page 7*

Care and Cleaning

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

WARNING

- Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.
- The bandage is intended for the individual care of the patient.
 - In case of improper use, product liability is excluded.
 - A combination with other products must be discussed in advance with the doctor.
 - If the symptoms increase or you notice abnormal changes, please consult your doctor.
 - Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.

Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.

5