SCHAPER

Manufacturer

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The Schapfix Genu Knee brace provides ideal support for the knee joint and offers great comfort when being worn. The brace relieves strain on the knee cap, has a pain relieving action, comfortably stabilises the knee joint and ensures effective compression.

Applying the Bandage *See page 7*

Care and Cleaning

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

WARNING

- Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.
- The bandage is intended for the individual care of the patient.
 - In case of improper use, product liability is excluded.
 - A combination with other products must be discussed in advance with the doctor.
 - If the symptoms increase or you notice abnormal changes, please consult your doctor.
 - Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.

Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.