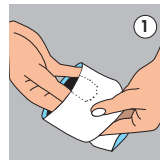
Manufacturer

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The Malleo Ankle support is an active ankle bandage which can be used after spraining or twisting of the ankle, surgery or when suffering from osteoarthritis. The highly elastic knit work, made of breathable material, ensures an effective compression and a comfortable fit (the support will feel like a sock), and it activates the blood circulation. It can be worn during rehabilitation, sports and daily activities.

Applying the Bandage *See page 7*

Care and Cleaning

Wash in warm water (by hand) with a mild detergent, rinse thoroughly and hang dry. Do not wash in the machine or dry clean. Do not tumble dry.

WARNING Do not wear on open wounds or if you have a tendency to skin disorder, allergic reaction or circulatory disorder.

- The bandage is intended for the individual care of the patient.
- In case of improper use, product liability is excluded.
- A combination with other products must be discussed in advance with the doctor.
- If the symptoms increase or you notice abnormal changes, please consult your doctor.
- Please do not allow the bandage to come into contact with fatty and acidic agents, ointments and lotions.

Not a substitute for medical care. Always consult a doctor for injury diagnosis and treatment.