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Ankelledsbandage med pelotte til bløddelskompression og let stabilisering.

Indikationer:

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Intended use:

Ankle brace with pad for soft tissue compression and light stabilisation.

Bandage

Indications:

Mild instabilities
Ankle joint sprain
Postoperative and posttraumatic irritation of the ankle joint
Arthrosis
Swelling and joint effusions

Contraindications:
Currently unknown.

Bandage

Application instruction:

Fold the upper part of the brace outwards until the base of the silicone pad becomes visible (1). Pull the brace over the foot (2). Fold the upper part of the brace back again and pull the brace upwards if required. Position the brace so that the heel sits correctly in the heel area. You can use the white stocking to help you put on the bandage. First pull it over the foot (3). Then pull the bandage over the foot and the stocking (4). Once the bandage is put on correctly, grab the stocking in the area of the toes and pull it out from under the bandage (5).

Bandage

Application note / Caution:

- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
- This product is intended for use on a single patient only.
- If necessary, the help of a second person can be beneficial for the correct application of the product.
- Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
- The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
- Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
- Do not put on the product too tightly, as this may affect the blood circulation.
- If symptoms persist or worsen, please consult your medical doctor.
- Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
- If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
- Do not make any changes to the product.
- Simultaneous use with other products only after consultation with your doctor.

Bandage

Care instructions:

The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Bandage

Material composition:

Polyamide (PA), Elastane (EL)

Bandage

Storage instructions:

Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Bandage

Disposal note:

Disposal via household waste is possible.

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