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EN

Intended use:
Back bandage for supporting and stabilising the lumbar spine.

Indications:
Pain in the lumbar spine region
Low back pain
Mild spondylarthrosis

Contraindications:
Currently unknown.

Application instruction:
Use the Velcro fastener to adjust the position of the pad. When positioning the pad, make sure that it rests centrally on the bandage. Open the Velcro fasteners of all straps. To make subsequent application easier, both straps can be fastened to the two upper straps. The two upper straps can be fastened to the two Velcro attachment points next to the basic closure. Arrange the bandage around the waist so that the pad is positioned in the middle of the back and the side with the broader straps points to the foot. Insert the fingers in the hand straps on the right and left closure side. Evenly pull the two closure sides forward and close the two sides on top of each other. For this, first press the left closure side against the abdomen and then fasten the right closure side using the Velcro fastener (1+2+3). When fastening the two closure sides, first pull the fingers out of the left hand strap and then out of the right hand strap. Next, grab the two narrow upper straps, pull these forward while applying even tension and use the Velcro fastener to close the two straps on the basic closure (4+5). Lastly, grab the two straps of the additional fixation, pull these forward while applying even tension and use the Velcro fastener to close the two straps on the basic closure (6). Make sure that all hook-and-loop fastening areas are fully covered by the product and cannot come into contact with clothing or the skin.

- Application note / Caution:**
- Do not use the product on open wounds. In the following cases you should consult your doctor before use: Skin diseases, lymphatic drainage disorders, sensory disorders, circulatory disorders, unclear soft tissue swelling.
 - This product is intended for use on a single patient only.
 - If necessary, the help of a second person can be beneficial for the correct application of the product.
 - Only by choosing the correct size and variant, as well as the correct application, an optimal function can be achieved.
 - The product should no longer be used if it is damaged or heavily worn, or if existing joints and fasteners no longer function properly.
 - Make sure that existing Velcro fasteners do not cause damage to clothes or other fabric parts.
 - Do not put on the product too tightly, as this may affect the blood circulation.
 - If symptoms persist or worsen, please consult your medical doctor.
 - Please inform the manufacturer and the competent authority in your country about serious incidents with this product.
 - If you are unclear or have questions about the product and its handling, please consult your doctor or qualified personnel.
 - Do not make any changes to the product.
 - Simultaneous use with other products only after consultation with your doctor.

Care instructions:
The product can be cleaned by hand with water and a mild detergent. The plastic and metal parts of the product (if existent) should not be washed and must be removed if possible. These parts can be cleaned with a damp cloth. Close existing Velcro fasteners before washing to avoid damaging other laundry items. Allow to air dry. Avoid direct heat sources and do not machine dry or iron. Make sure the product is completely dry before using it again. Do not dry clean and do not use bleach, solvents, or fabric softeners. Fatty or acidic agents, ointments or lotions can damage the material.

Material composition:
Polyamide (PA), Polyester (PES), Elastane (EL)

Storage instructions:
Store the product in a dry place and protect it from heat, direct sunlight and moisture.

Disposal note:
Disposal via household waste is possible.

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